
ACH IB-T System Digital Library

3Shape Manual

■ Download the library and unzip the compressed file.

1. Unzip the downloaded file

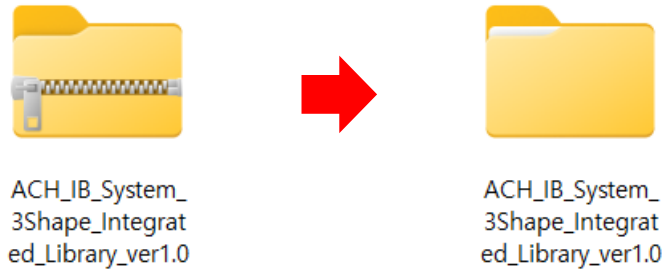2. Library files included in the unzipped folder.

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

■ Import Library

1. Click on the '3Shape Dental System Control Panel' icon



2. “Click” Tools → Import/Export

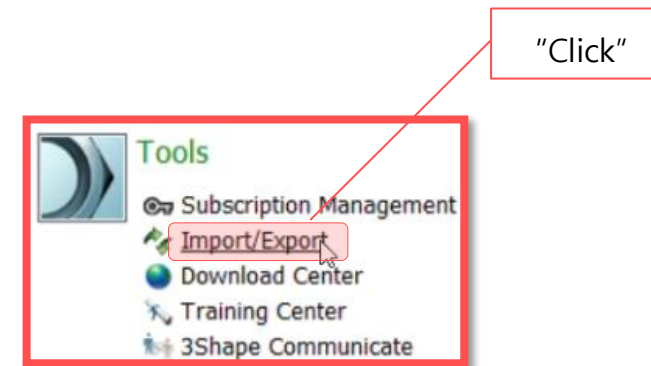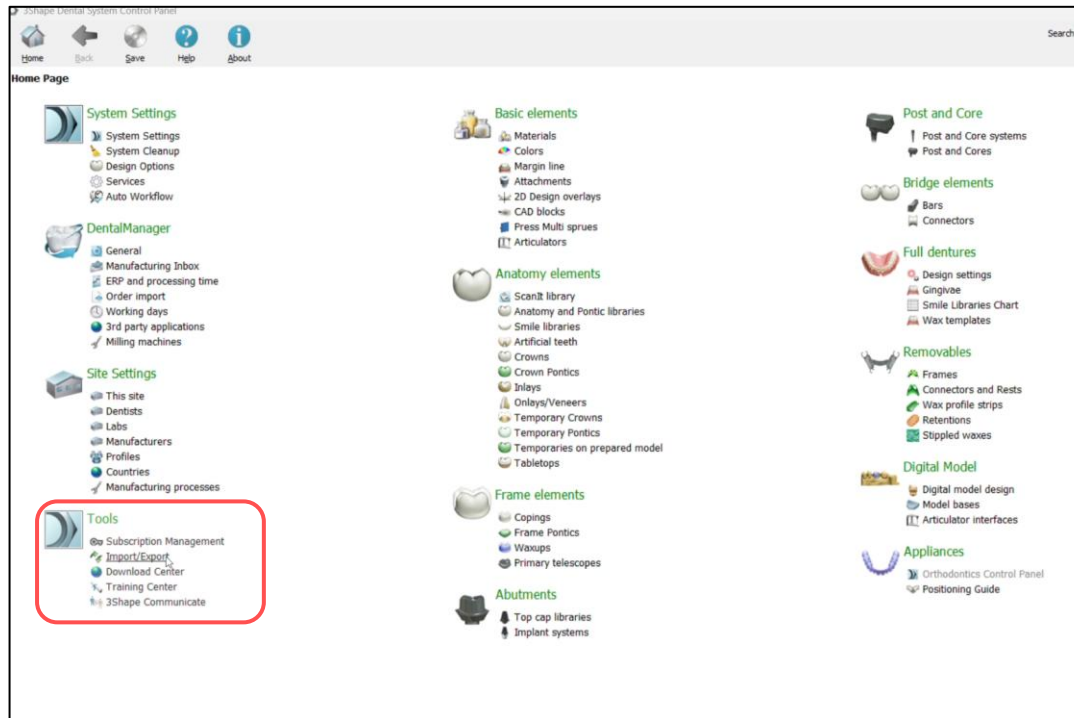■ Import Library

3. "Click" the Import materials

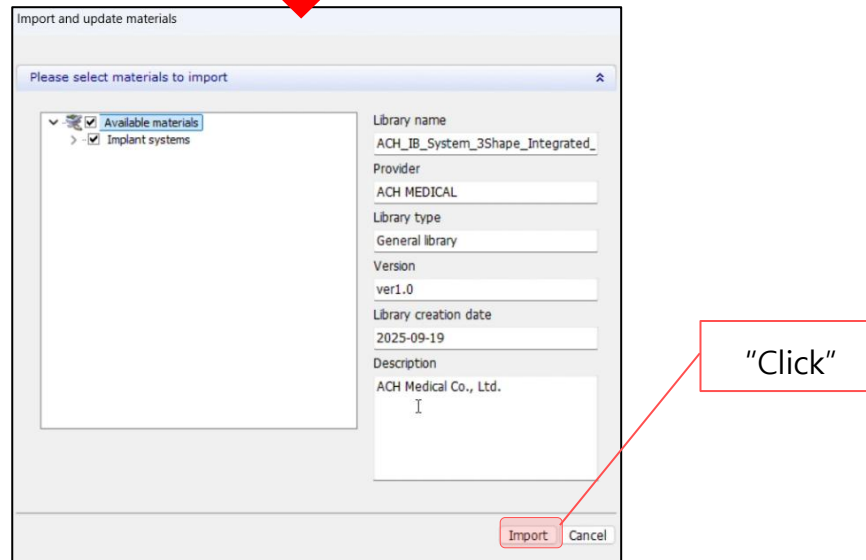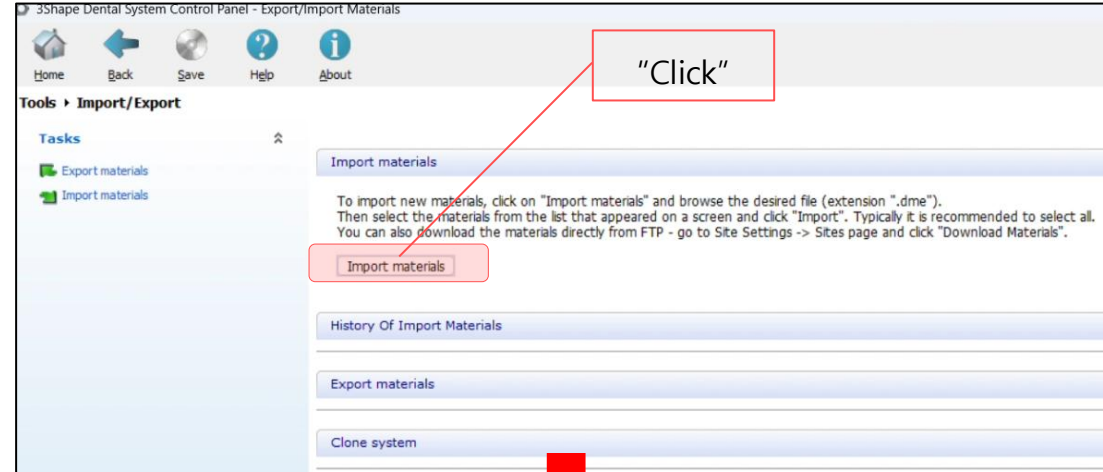■ Import Library

4-1. Select the library if you want to import “all files of the IB System”

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① All library files

4-2. Select the library if you want to import “individual IB System files”

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④ IB-T MUA(Multi unit)

⑤ IB-T PINK


■ Design : Customized Abutment (Premill)

1. Click the “3Shape Dental Designer” icon


2. Select “System” in the order window.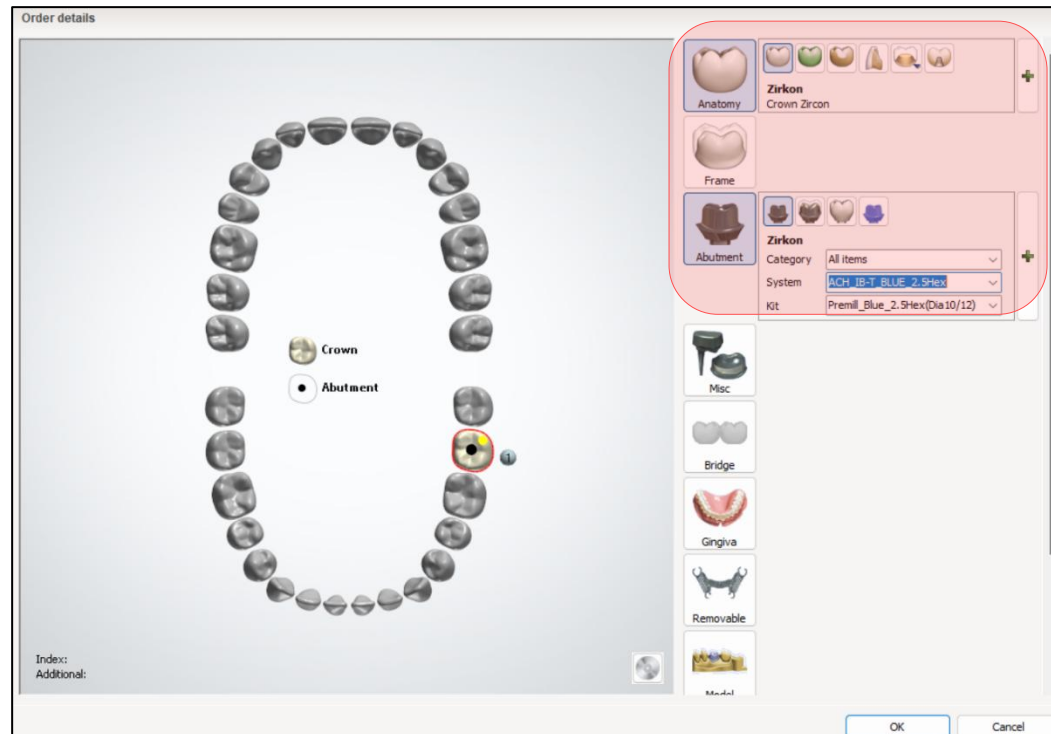

■ Design : Customized Abutment (Premill)

3. Select the scan body that matches the system specifications.

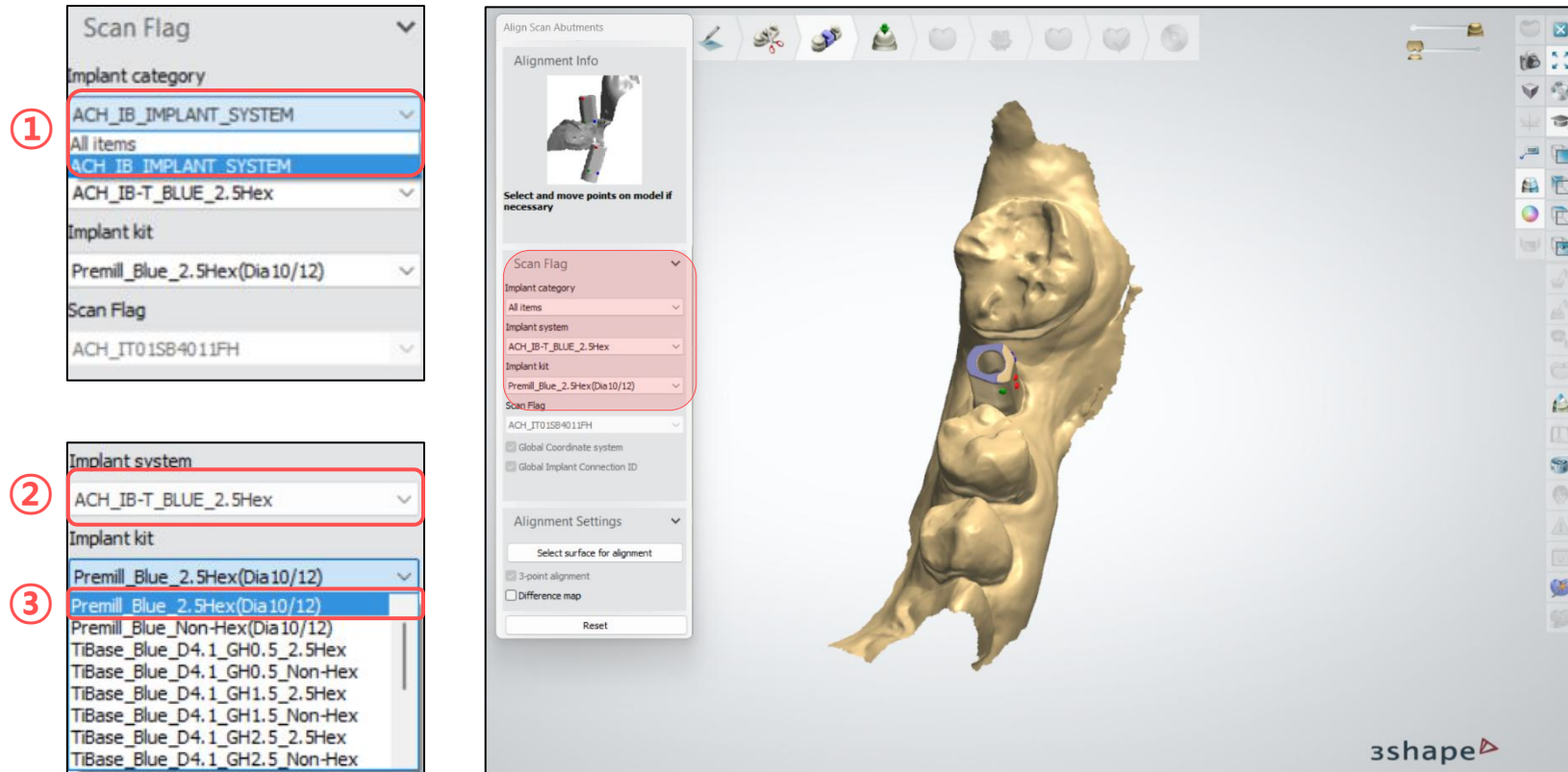※ If the scan body was selected incorrectly in the order form, you can reselect it at this step

- Design : Customized Abutment (Premill)


4. Proceed with prosthetic design.

- Design : Customized Abutment (Premill)


5. Final stage of design.

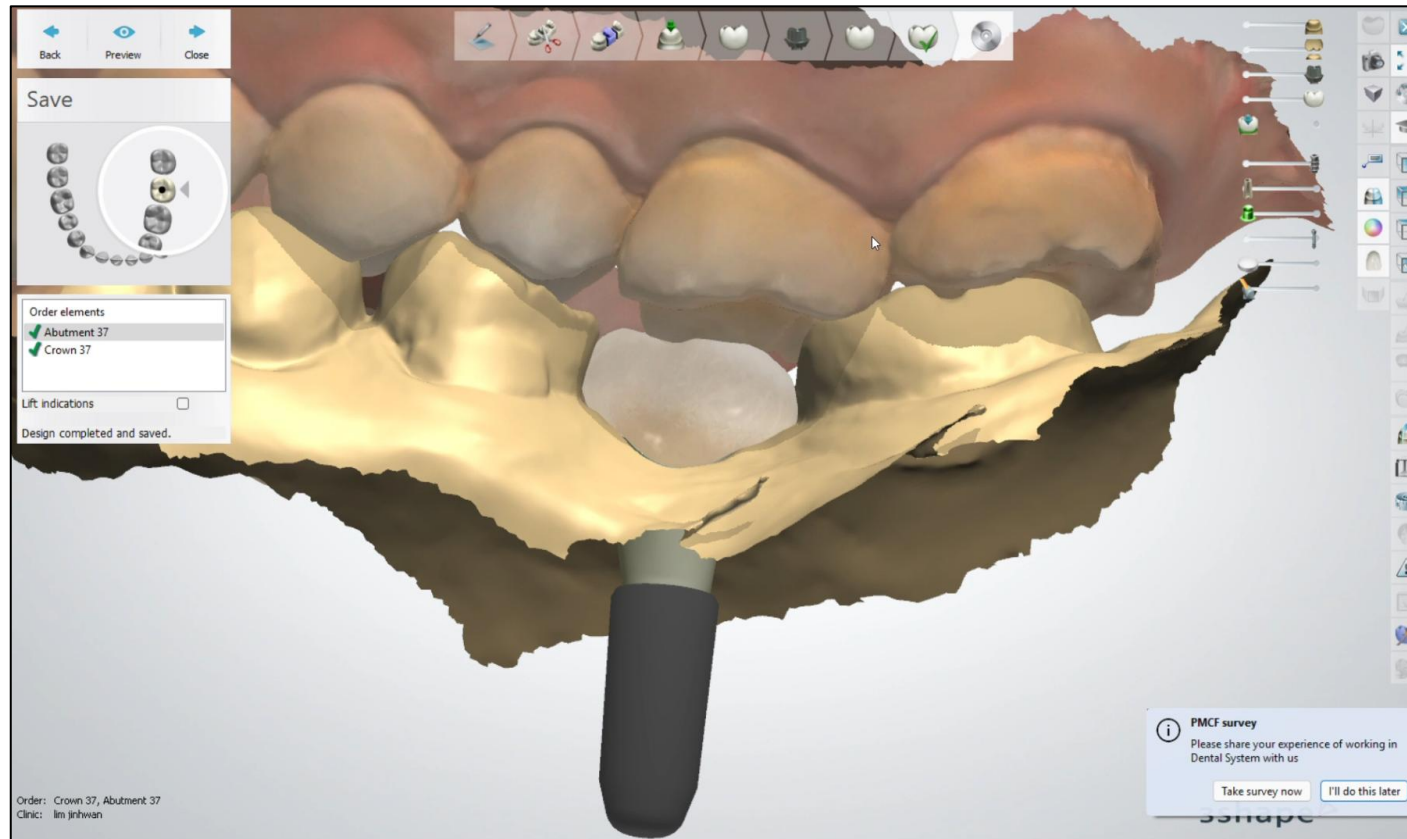
- Design : Link Abutment (Ti-Base)

1. Select "System" in the order window.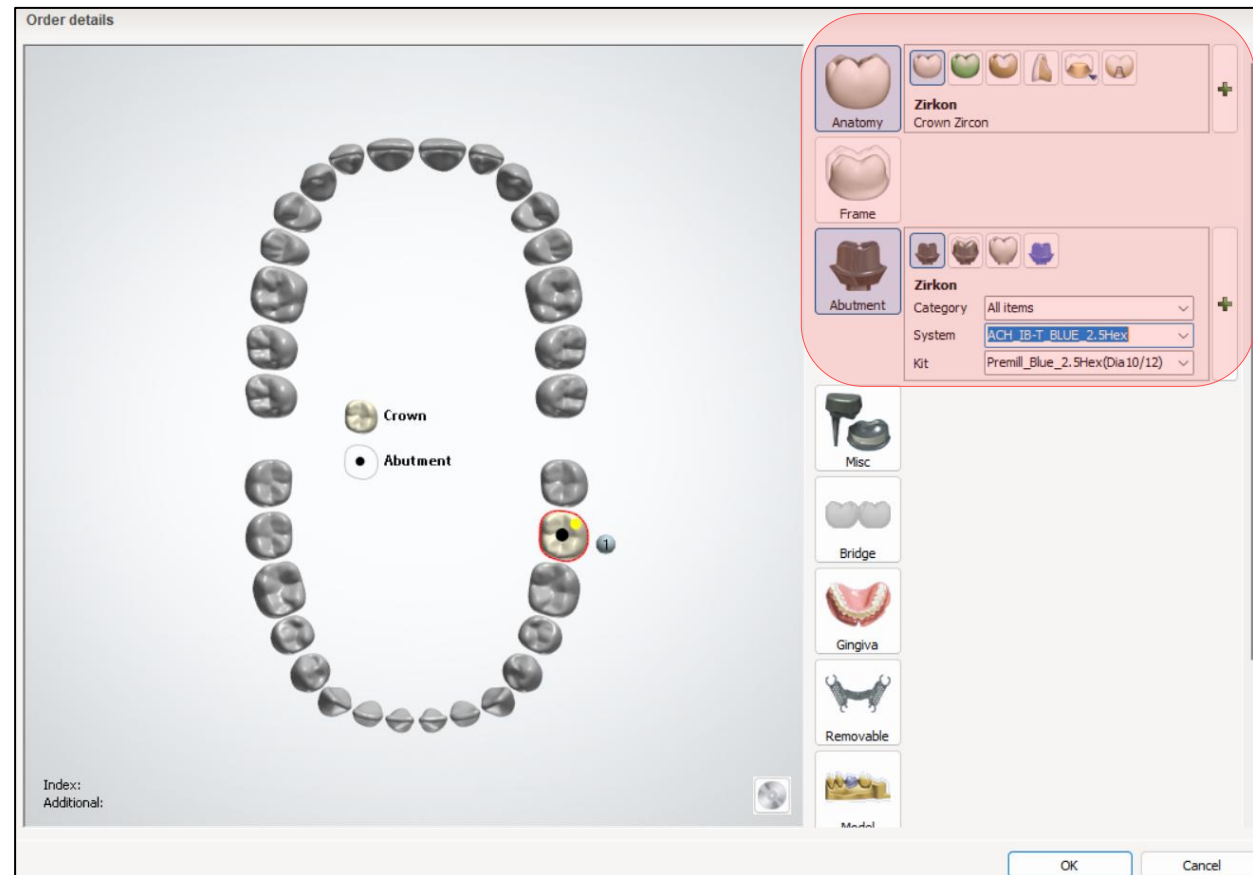

■ Design : Link Abutment (Ti-Base)

2. Select the scan body that matches the system specifications.

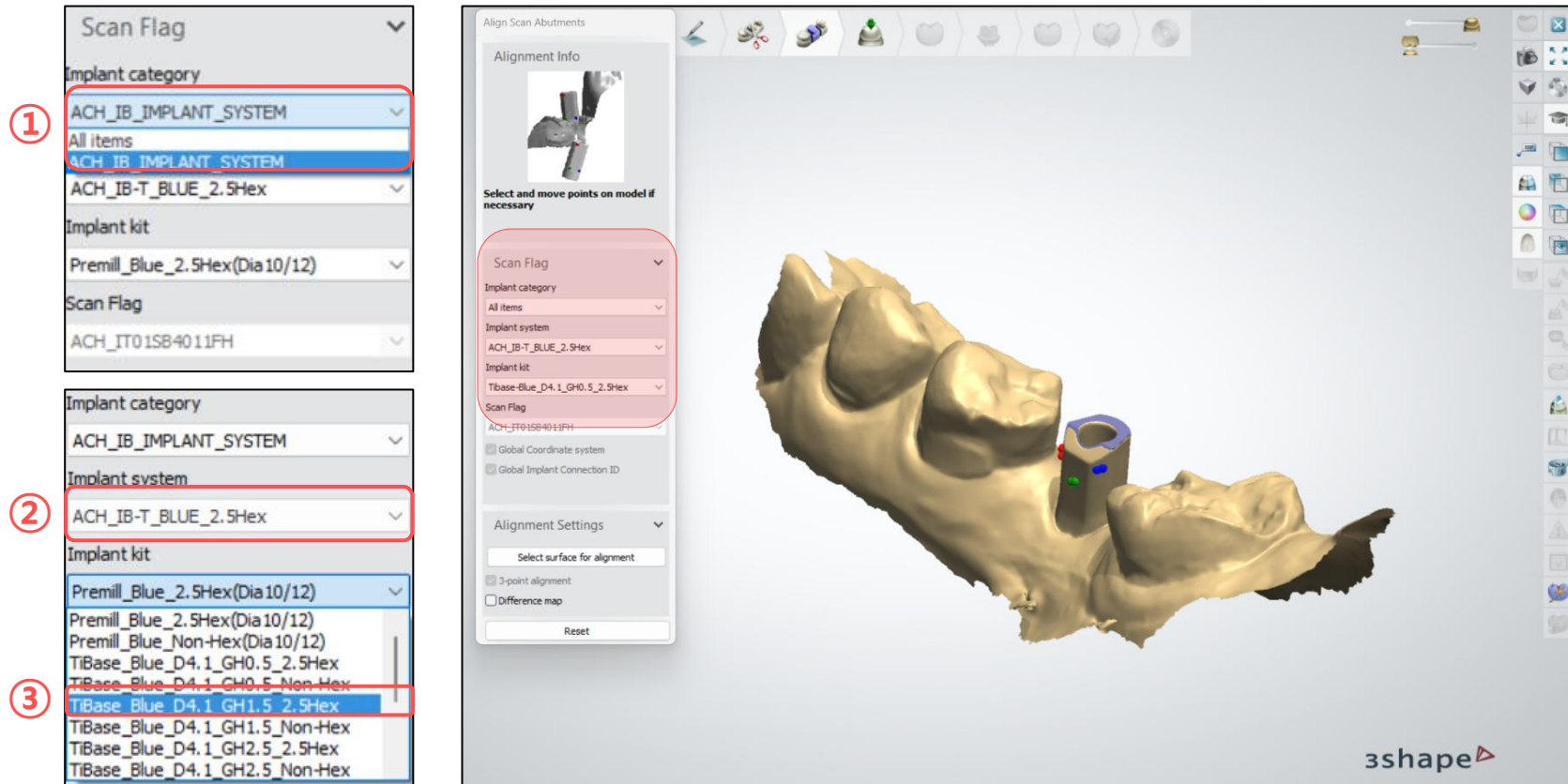
※ If the scan body was selected incorrectly in the order form, you can reselect it at this step.



- Design : Link Abutment (Ti-Base)

3. Proceed with prosthetic design.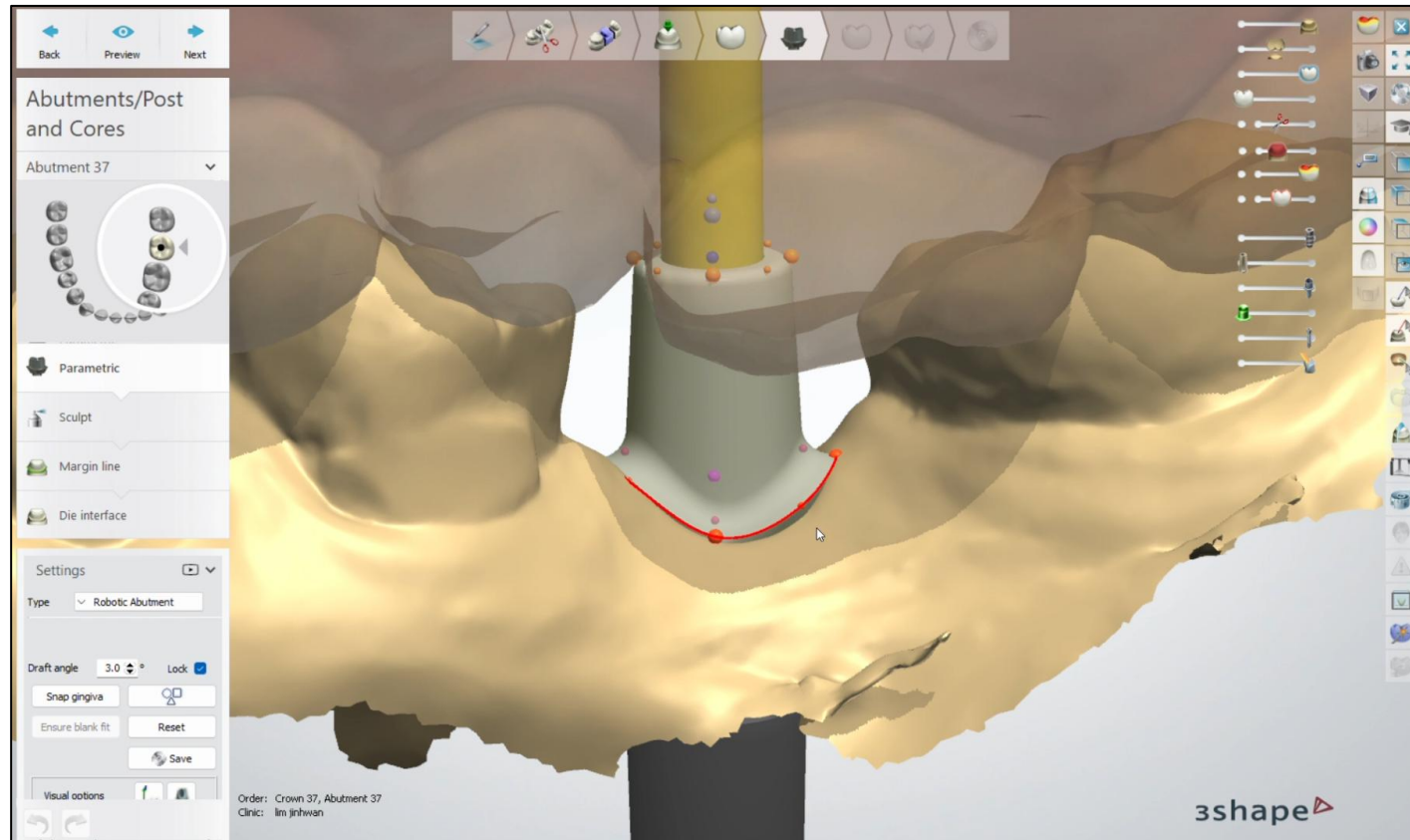

- Design : Link Abutment (Ti-Base)

4. Proceed with setting the cement gap.

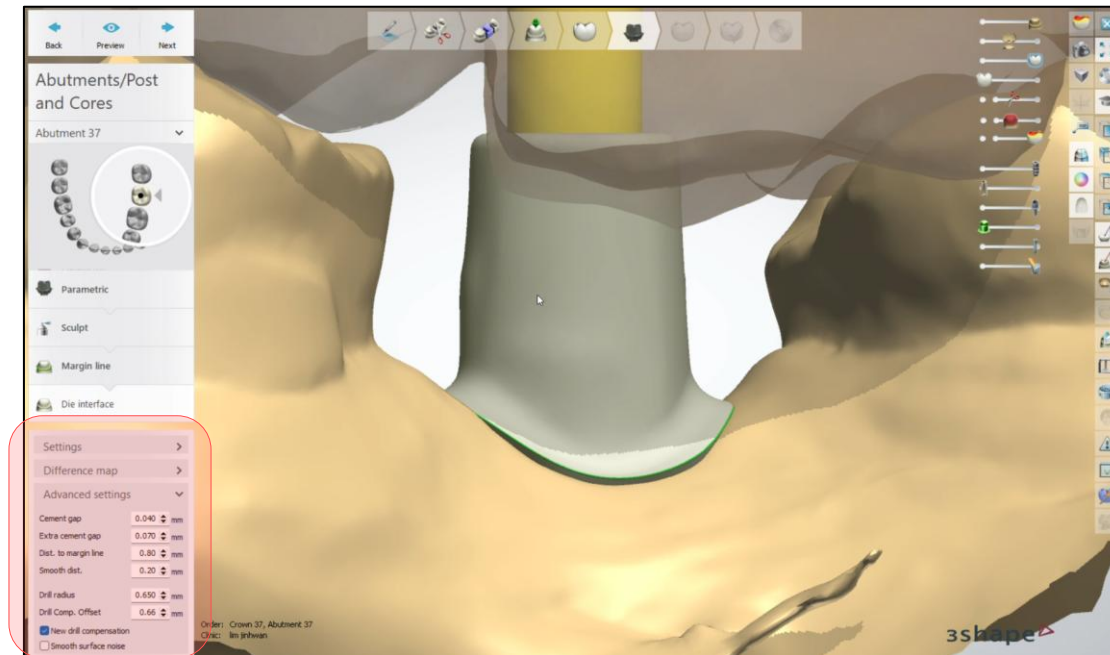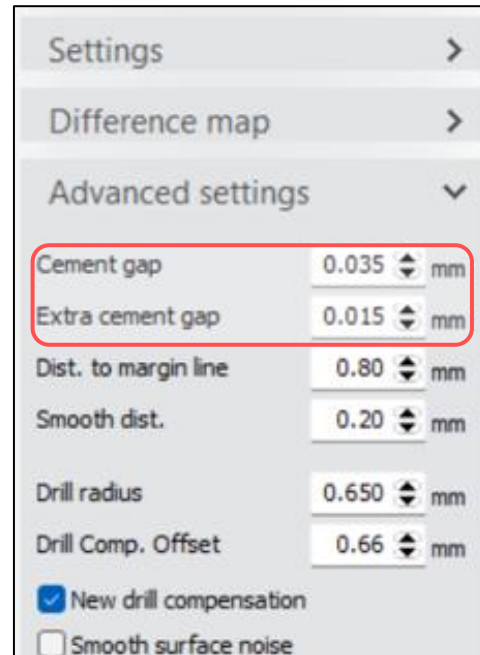
※ Cement Gap *Default value: 0.035

Set the cement gap and extra cement gap values for the internal surface of the crown.

- Design : Link Abutment (Ti-Base)

5. Final stage of design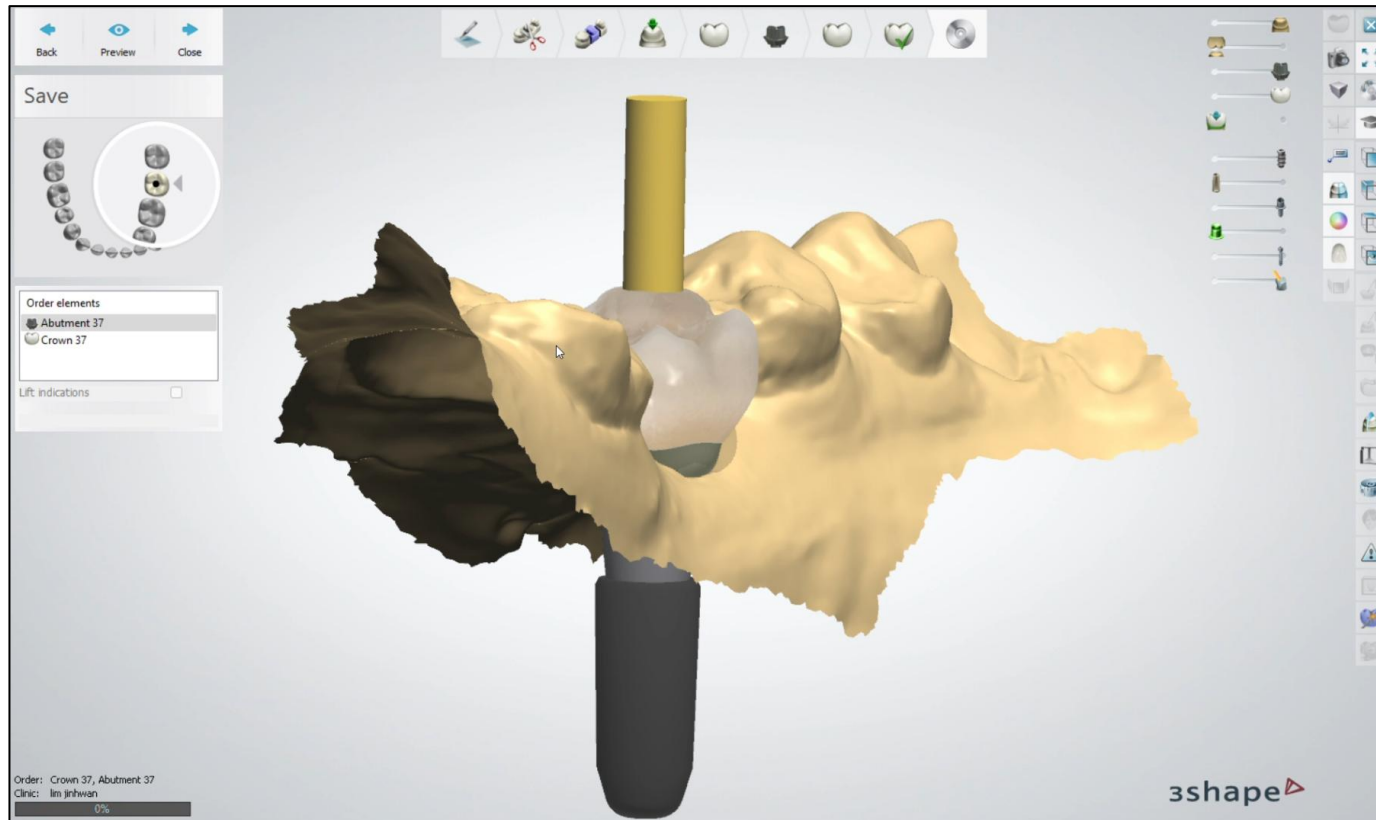

Thanks

Through continuous development
We will provide better service and products!

Achieving the future with ACH Medical